



# B49 BUBBLEGUM APPLICATOR

Applying your Bubblegum tags will be easier and quicker with our improved applicator design:

**NEW  
IMPROVED  
DESIGN**

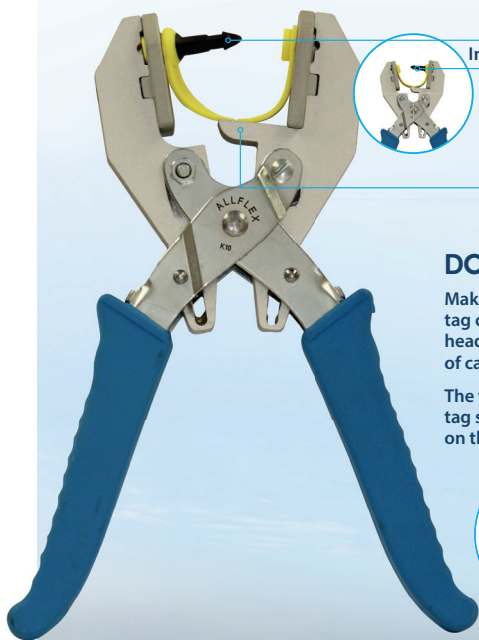

Insert tag either way

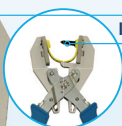

We've added a support to ensure that the tag is held firmly in place

## DON'T FORGET

Make sure to apply the tag close to the animal's head avoiding the ridges of cartilage and veins.

The female part of the tag should always sit on the inside of the ear.



 0845 600 9070

 [www.allflex.co.uk](http://www.allflex.co.uk)

The Allflex UK Group Ltd,  
1 Greencroft Industrial Park,  
Stanley, DH9 7YA

**Allflex**<sup>®</sup>

Animal Intelligence, for life