

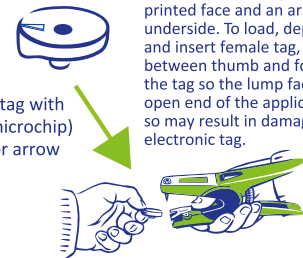
# FLIP PIN AND UNIVERSAL TOTAL TAGGER APPLICATORS

ELECTRONIC CATTLE TAGS

VISUAL TAGS AND ELECTRONIC SHEEP TAGS

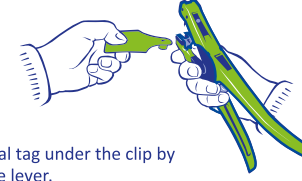
**1**  Ensure that the black insert is removed from the jaws of the Universal & Flip Pin applicators.

**1**  Ensure that the black insert is inserted into the jaws of the Universal & Flip Pin applicators.

**2**  The female NLIS tag has a lump on its printed face and an arrow on the underside. To load, depress spring clip and insert female tag, holding the lump between thumb and forefinger. Position the tag so the lump faces out of the open end of the applicator. Failure to do so may result in damage to the electronic tag.

Female tag with lump (microchip) indicator arrow

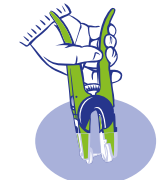
Lump faces out from applicator

**2**  Place the visual tag under the clip by depressing the lever.

It is recommended that adult cattle are restrained in a head bail for tagging.

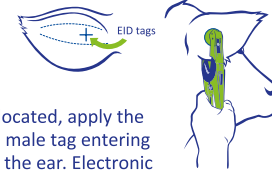
**3**  Slip male button completely onto application pin.

**3**  Slip male tag completely onto application pin.


**4**  Dip jaws of applicator holding tag into antiseptic or disinfectant solution.

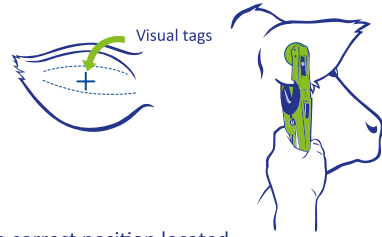
**NB:** Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.

**NB:** The correct applicator must be used when applying NLIS ear tags.

**5**  With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear. Electronic tags should be attached in the centre of the inside third of the ear (as shown above).

The electronic tag must be applied to the RIGHT (off-side) ear as per the illustration above.



**5**  Visual tags

With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear.

Note: visual tags and electronic sheep tags = INSERT IN  
 Electronic cattle tags = INSERT OUT



Flip Pin Applicator



Universal Total Tagger

Proven & Trusted

7 Nortjé Street, PO Box 125,  
 Strand, Cape Town, 7139  
 T: 021 854 8877 | F: 086 773 8880  
 info@allflexsa.com



Identify the Difference