



Allflex Livestock Intelligence

## UTT3S Applicator

The UTT3S Applicator brings the application of animal ear tags and devices to new heights for simplicity, smoothness, and safety. With innovative mechanisms that greatly reduce required application force and a unique ergonomic design, the UTT3S is in the highest class of animal device applicators. It simplifies device application and management, as the same applicator can be used for attaching varied visual ID, electronic ID and monitoring devices to a wide variety of species.



Livestock Intelligence™

MSD Animal Health Intelligence



- **Simple to use** – A lightweight applicator with an easy-squeeze mechanism that enables the application of devices with very little force. The ergonomic design and low application force together allow comfortable use by almost any adult. A spare pin is conveniently located in the handle of the applicator, ensuring a pin is easily accessible when needed.
- **Smart, versatile design** – The UTT3S allows operators to attach two-piece visual and electronic identification devices to cattle, sheep, goats and swine of all ages. The applicator also applies Allflex monitoring ear tags to cattle. A simple pin change enables efficient switching between device types to aid in a wide variety of on-farm workflows. The robust design enables long-term use in the harshest farm environments.
- **Smooth and safe for all** – Enhanced safety features and low application force protect users' hands and enable them to comfortably apply devices to multiple animals, in line with the growing demands of on-farm labor. The innovative design enables fast and quiet application, for maximum animal comfort and welfare while devices are being applied. Reduced pinch points on the applicator keep users safe while applying tags and devices to animals' ears.

### Supported Tags

The UTT3S Applicator can be used to apply most types of Allflex two-piece ear tags:<sup>1</sup>

Allflex two-piece visual identification tags (VID)

Allflex two-piece electronic identification tags (EID)

Allflex monitoring ear tags

<sup>1</sup>The UTT3S Applicator does not work with piglet tags, lightweight EID button tags for cattle and swine, AXA tags, or one-piece tags.

## About Allflex Livestock Intelligence

Allflex Livestock Intelligence is the world leader in the design, development, manufacturing and delivery of solutions for animal identification, monitoring and traceability. Our data-driven solutions are used by farmers, companies and countries to manage hundreds of millions of animals worldwide. By putting intelligent, actionable management information into farmers' hands, our solutions empower them to act in a timely manner to safeguard their animals' health and wellbeing, while achieving optimal production outcomes for a healthy food supply. Allflex Livestock Intelligence is a livestock portfolio of digital products within MSD Animal Health.

Through its commitment to the Science of Healthier Animals®, MSD Animal Health offers veterinarians, farmers, pet owners and governments one of the widest ranges of veterinary pharmaceuticals, vaccines and health management solutions and services as well as an extensive suite of digitally connected identification, traceability and monitoring products. MSD Animal Health is dedicated to preserving and improving the health, wellbeing and performance of animals and the people who care for them.

Copyright© 2021 Merck & Co., Inc., Kenilworth, NJ USA, and its affiliates. All rights reserved. The information contained herein is subject to change without notice. The only warranties for Allflex's products and services are set forth in the express warranty statements accompanying such products and services. Nothing herein should be construed as constituting an additional warranty. Allflex Livestock Intelligence shall not be liable for technical or editorial errors or omissions contained herein.